

Prevantics® Device Swab

The first and ONLY 3.15% Chlorhexidine Gluconate/70% Isopropyl Alcohol solution authorized by the FDA to disinfect needleless access sites prior to use.



STEP 1

open
package




STEP 2

5 second
scrub
time



STEP 3

5 second
dry time

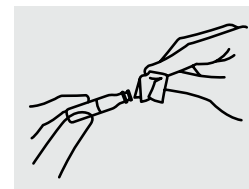

INSTRUCTIONS FOR USE

- Tear open swab packet and remove swab.
- Do not unfold swab.
- Prior to injections, apply swab to the needleless access site of the device by holding swab between thumb and index finger and scrub using repeated back-and-forth strokes for 5 seconds.
- Allow to air dry for 5 seconds.
- Do not blot or wipe dry.
- Discard after single use.



REORDER # _____ REPLACING ORDER # _____

PDI

Two Nice-Pak Park, Orangeburg, NY 10962-1376 + 1-800-999-6423
+ Phone: (845) 365-1700 + Fax: (845) 398-5347 + pdihc.com

©2014 Professional Disposables International, Inc. All rights reserved.
PDI10147832

