



INTERVENTIONAL CARE

PATIENT CARE

ENVIRONMENT OF CARE

Two Nice-Pak Park
Orangeburg, New York 10962
pdihc.com

March 22, 2018

Dear Valued Customer:

We understand that equipment compatibility is an important factor when considering a surface disinfectant. Therefore, our R&D team has conducted compatibility testing of our **Sani-Cloth®** brand germicidal disposable wipes on common surface materials found within the healthcare environment.

The results can be found on the Material Reference Guide enclosed with this letter. Please use the results as a general guide rather than a guarantee of compatibility, and always refer to the surface or equipment manufacturer's guidelines for approved disinfecting instructions.

For more information about **Sani-Cloth** brand wipes, please contact your local PDI representative or call 1-800-999-6423.

Sincerely,

Arpan Desai
Compatibility Manager

ABOUT PDI

At PDI, we never forget who we are all working for. That's why we offer a broad range of evidence-based, market leading Interventional Care, Environment of Care, and Patient Care solutions, all designed to help reduce preventable infections, control healthcare costs, and ultimately help save lives. It's all part of our passion to *Be The Difference®*, every day. To learn more, visit www.pdihc.com.



Sani-Cloth® Materials Compatibility Reference Guide

Surface Material		Super Sani-Cloth®	Sani-Cloth® Plus	Sani-Cloth® Bleach	Sani-Cloth® AF3	Sani-Cloth® Prime
Metals	Aluminum Alloy 6061	●	●	●	●	●
	Brass Alloy 260	●	●	●	●	●
	Chrome Plated Metal	●	●	●	●	●
	Copper Alloy 110	●	●	●	●	●
	Nickel	●	●	●	●	●
	Nickel Plated Metal	●	●	●	●	●
	Stainless Steel 304	●	●	●	●	●
	Stainless Steel 316	●	●	●	●	●
	Steel, SAE 1008/1010	●	●	●	●	●
	Titanium	●	●	●	●	●
Zinc	●	●	●	●	●	
Polymers	Acrylic	●	●	●	●	●
	Acrylonitrile Butadiene Styrene (ABS) Cicolac™ MG47	●	●	●	●	●
	Polybutylene Terephthalate (PBT) Valox™ 215 HPR	●	●	●	●	●
	Polycarbonate	●	●	●	●	●
	Polyester	●	●	●	●	●
	Polyethylene	●	●	●	●	●
	Polypropylene	●	●	●	●	●
	Polystyrene (high impact)	●	●	●	●	●
	PVC tubing	●	●	●	●	●
Rubber	Santoprene™	●	●	●	●	●
	Silicone Rubber	●	●	●	●	●
Ceramic	Glazed Ceramic Tile	●	●	●	●	●
Glass	Soda Lime Glass	●	●	●	●	●

Use these results as a general guide rather than a guarantee of compatibility. Please refer to the surface or equipment manufacturer's guidelines for approved disinfecting instructions.

Cicolac™ and Valox™ are trademarks of the SABIC Global Technologies B.V.

Santoprene™ is a trademark of Exxon Mobil Corporation

Legend	
●	Compatible
●	Moderate (Visual) Degradation
●	Severe (Physical) Degradation