

# GENERAL GUIDELINES FOR USE

## Follow the steps below for application



**Item No. S40750**  
**Peel Apart**



**Item No. S42850 and S32450 (kit products)**  
**Tear Open**

**1. For Item No. S40750:** Peel apart the pouch to expose the swabstick. Remove the swabstick from the package. Do not touch the foam tip.

**For Item No. S42850 (compact swabstick - kit product) and Item No. S32450 (swabstick - kit product):** Grasp the pouch at the notch and tear open straight across. Remove the swabstick from the package. Do not touch the foam tip.



**2. For dry sites:** Scrub repeatedly back and forth for 15 seconds with each side of the applicator for a total of 30 seconds.

**For moist sites:** Scrub repeatedly back and forth for 1 minute with each side of the applicator for a total of 2 minutes.

**Maximum treatment area for one swabstick:**  
dry site - 4 x 4 in (10 x 10 cm),  
moist site - 3 x 5 in (7.5 x 12.5 cm)



**3. Do not blot or wipe dry.**

**For dry sites:** allow the prepped area to air dry for 30 seconds.

**For moist sites:** allow to air dry for approximately 1.5 minutes.

**4. Discard after single use.**  
Do not use with electrocautery procedures.



**SCAN HERE**  
**TO WATCH VIDEO**