


# +Prevantics®

3.15% Chlorhexidine Gluconate (CHG) and  
70% Isopropyl Alcohol (IPA)

Device


SWAB

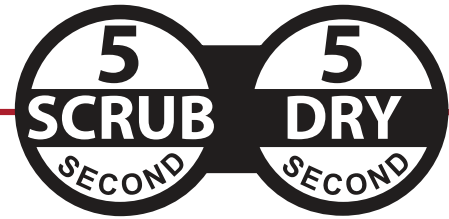

## GENERAL GUIDELINES FOR USE




1. Look for the words “tear below” (found on back side of packet) and tear downward diagonally to retrieve swab.

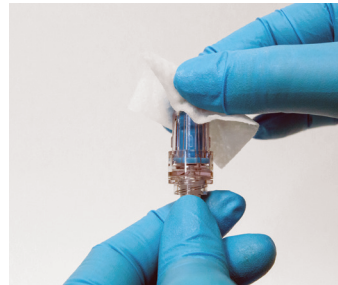


2. Prior to access, apply swab to the needleless access site of the device by holding swab between thumb and index finger and scrub using repeated back and forth strokes for 5 seconds.



3. Allow to air dry for 5 seconds. Do not blot or wipe dry.

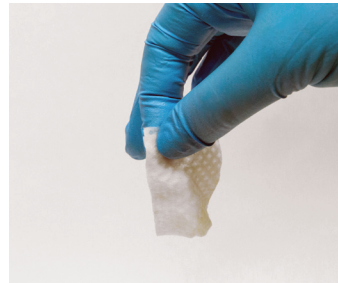

4. Discard after single use.

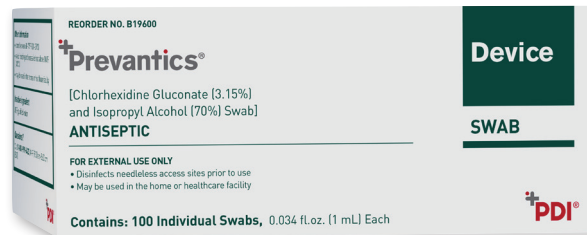

+PDI®  
BE THE DIFFERENCE™