Sani-Cloth®

GERMICIDAL DISPOSABLE WIPE
LINGETTE GERMICIDE JETABLE

AF3

LARGE WIPE
GRANDE LINGETTE

BACTERICIDAL, TUBERCULOCIDAL, VIRUCIDAL* IN 3 MINUTES
BACTÉRICIDE, TUBERCULOCLIDE, VIRUCIDE* EN 3 MINUTES

ACTIVE INGREDIENTS:

n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides.....	0.14%
n-Alkyl (60% C ₁₂ , 30% C ₁₄ , 5% C ₁₂ , 5% C ₁₄) dimethyl benzyl ammonium chlorides.....	0.14%
OTHER INGREDIENTS	99.72%
TOTAL	100.00%

Does not include the weight of the wipe.

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chloures de n-alkyl (68% C ₁₂ , 32% C ₁₄) alkylidiméthyl (éthylbenzyl) ammonium.....	0.14%
chloures de n-alkyl (60% C ₁₂ , 30% C ₁₄ , 5% C ₁₂ , 5% C ₁₄) alkylidiméthyl (éthylbenzyl) ammonium.....	0.14%
AUTRES INGRÉDIENTS	99.72%
TOTAL	100.00%

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Net Wt. 1.0 lb 11.4 oz (777g) / Poids net : 1.0 lb 11.4 oz (777 g)

REORDER NO. / N° DE RE-COMMANDE P13872

Effective Against / Efficace contre :

BACTERIA • *Bordetella bronchiseptica* • *Bordetella pertussis* • *Burkholderia cepacia*
• *Campylobacter jejuni* • *Enterobacter aerogenes* • *Escherichia coli* • *Escherichia coli* O157:H7
• *Klebsiella pneumoniae* • *Listeria monocytogenes* • *Proteus vulgaris* • *Pseudomonas aeruginosa*
• *Salmonella enterica* • *Serratia marcescens* • *Shigella dysenteriae* • *Staphylococcus aureus*
• *Streptococcus pyogenes* • *Vibrio cholera* • *Yersinia enterocolitica*

MULTI-DRUG RESISTANT BACTERIA • *Acinetobacter baumannii*, Multi-Drug Resistant
• Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARSA
NRS123 (Genotype USA 400) • Community Acquired Methicillin Resistant *Staphylococcus*
aureus (CA-MRSA) NARSA NRS384 (Genotype USA 300) • ESBL Resistant *Escherichia coli*
• *Escherichia coli* - NDM-1 Positive CDC 1001728 • ESBL Resistant *Klebsiella pneumoniae*
• *Klebsiella pneumoniae* - Carbapenem Resistant • *Klebsiella pneumoniae* - NDM-1 Positive
CDC 1000527 • Methicillin Resistant *Staphylococcus aureus* (MRSA) • *Streptococcus*
pneumoniae - Penicillin Resistant • Vancomycin Resistant *Staphylococcus aureus* (VISA)
NARSA VRS1 • Vancomycin Resistant *Enterococcus faecalis* (VRE)

VIRUSES • *Adenovirus type 5, Strain Adeno75 • *Avian Influenza A H5N1 virus
• *Cytomegalovirus, Strain AD-169 • *Herpes simplex virus type 2, Strain G • *Human
Coronavirus, Strain 229E • *Influenza A virus/Hong Kong Strain • *Influenza B virus, Strain
B/Hong Kong/5/72 • *Respiratory syncytial virus (RSV), Strain Long • *Rotavirus Strain WA
• Kills Pandemic 2009 H1N1 Influenza A Virus

BLOODBORNE PATHOGENS: *Hepatitis B virus (HBV)-Duck HBV • *Hepatitis C virus
(Human) [HCV]-Bovine Diarrhea virus • *HIV-1 (AIDS virus) Strain HTLV-III_g

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Mycobacterium (TB): *Mycobacterium bovis* - BCG (TB)

8BP-13800

*KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRECLEANED EXTERNAL SURFACES OF ULTRASOUND TRANSDUCERS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in three (3) minutes at room temperature (68°-77°F) in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV). **AREAS OF USE: Hospital and Healthcare Settings:** Ambulances, Ambulatory Surgical Centers (ASC), Clinics, Dental Offices, Dialysis Clinics, Healthcare Settings, Home Health Care, Hospitals, Laboratory, Nursing homes, Physicians' offices, Radiology and Rehabilitation, Critical Care Areas: CCU, Emergency Rooms, E.R., ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU) and Surgery and Surgical Intensive Care Unit (SICU), Hospital, Healthcare, and Critical Care Use Sites: May be used on hard nonporous surfaces of: bed railings, blood glucose meters, cabinets, carts, chairs, counters, dental unit instrument trays, exam tables, gurneys, infant incubators, infant warmers, isolettes; IV poles; operating room tables and lights; physical therapy (PT) equipment surfaces, stethoscopes, stretchers, tables, telephones; toilet seats; and hard nonporous outside surfaces of: amalgamators and dental curing lights; diagnostic equipment; patient monitoring equipment; patient support and delivery equipment; and ultrasound transducers and probes. This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean and decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection. **DIRECTIONS FOR USE:** It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **To Dispense Wipes:** Remove lid and discard inner seal from canister. Find to use wipe roll, remove first wipe for use, twist corner of next wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out at an angle. When not in use keep center cap of lid closed to prevent moisture loss. **CLEANING AND DEODORIZING:** Wipe surface with towelette until clean. Let air dry. For disinfecting hard nonporous surfaces only. **TO DISINFECT AND DEODORIZE:** Unfold a clean wipe and thoroughly wet surface. Allow surface to remain wet for three (3) minutes. Let air dry. If present, remove gross filth prior to disinfecting. **ULTRASOUND TRANSDUCERS AND PROBES:** For use to clean and decontaminate external surfaces of Ultrasound Transducers, scopes and probes prior to high-level disinfection (HLD). To clean and decontaminate prior to high-level disinfection (HLD): Pre-clean gel from external surface of ultrasound transducer, scope and probe by using a clean washcloth or germicidal wipe. Then, follow device manufacturer instructions for HLD. **SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS AND HEPATITIS C VIRUS OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS. Personal Protection:** When using this product, wear disposable protective gloves, protective gowns, face masks, and eye coverings when handling HIV-1 (AIDS virus), HBV or HCV infected blood or body fluids. **Cleaning Procedure:** All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal cloth. Open and unfold first germicidal cloth to remove heavy soil. **Disposal of Infectious Materials:** Dispose of used towelette in accordance with local regulations for infectious waste disposal. **Contact Time:** Use second germicidal cloth to thoroughly wet surface. Allow surface to remain wet three (3) minutes, let air dry. Although efficacy at a 30 second contact time has been shown to be adequate against HIV-1 (AIDS virus), this time is not sufficient for the other non-HIV organisms listed on this label. Therefore, a 3 minute wet contact time must be used for all listed non-HIV organisms. **PRECAUTIONARY STATEMENTS Hazards to humans and domestic animals. CAUTION:** Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using toilet. **FIRST AID:** Call a poison control center or healthcare professional for treatment advice. Have the product container or label with you when calling a poison control center or healthcare professional or going for treatment. **If in eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes; then continue rinsing eye. **STORAGE AND DISPOSAL:** Do not contaminate water, food, or feed by storage and disposal. **Storage:** Store in dry location. When not in use keep center cap of lid closed to prevent moisture loss. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Container Disposal:** Nonrefillable container. Do not reuse or refill this container. Offer for recycling. If recycling is not available, put in trash collection. ***TUE LE VIH-1 (VIRUS DU SIDA), LE VIRUS DE L'HEPATITE B (VHB) ET LE VIRUS DE L'HEPATITE C (VHC) SUR LES SURFACES/OBJECTS ENVIRONNEMENTAUX PRE-NETTOYES ET LES SURFACES EXTERIEURES PRE-NETTOYES DES TRANSDUCTEURS ULTRASON ET DES SONDES PRECEDEMMENT SOULEEES DE SANG/DE FLUIDES CORPORELS EN DEUX (2) MINUTES à température ambiante de (68° - 77° F) dans les établissements de soins de santé ou autres dans lesquels il existe une probabilité de salissure des surfaces/objets inanimés avec du sang ou des fluides corporels; et dans lequel les surfaces/objets susceptibles d'être souillés par du sang ou des liquides organiques peuvent être associés au potentiel de transmission du virus de**

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Professional Disposables International, Inc.