Sani-Cloth® AF3

GERMICIDAL DISPOSABLE WIPE
LINGETTE GERMICIDE JETABLE

EXTRA LARGE WIPE
TRÈS GRANDE LINGETTE

Effective Against / Efficace contre :

BACTERIA • *Bordetella bronchiseptica* • *Bordetella pertussis* • *Burkholderia cepacia* • *Campylobacter jejuni* • *Enterobacter aerogenes* • *Escherichia coli* • *Escherichia coli* 0157:H7 • *Klebsiella pneumoniae* • *Listeria monocytogenes* • *Proteus vulgaris* • *Pseudomonas aeruginosa* • *Salmonella enterica* • *Serratia marcescens* • *Shigella dysenteriae* • *Staphylococcus aureus* • *Streptococcus pyogenes* • *Vibrio cholera* • *Yersinia enterocolitica*

MULTI-DRUG RESISTANT BACTERIA • *Acinetobacter baumannii*, Multi-Drug Resistant • Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARS34 (Genotype USA 400) • Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARS384 (Genotype USA 300) • ESBL Resistant *Escherichia coli* • *Escherichia coli* - NDM-1 Positive CDC 1001728 • ESBL Resistant *Klebsiella pneumoniae* • *Klebsiella pneumoniae* - Carbapenem Resistant • *Klebsiella pneumoniae* - NDM-1 Positive CDC 1000527 • Methicillin Resistant *Staphylococcus aureus* (MRSA) • *Streptococcus pneumoniae* - Penicillin Resistant • Vancomycin Resistant *Staphylococcus aureus* (VISA) NARS VRS1 • Vancomycin Resistant *Enterococcus faecalis* (VRE)

VIRUSES • * Adenovirus type 5, Strain Adenov 75 • * Avian Influenza A H5N1 virus • * Cytomegalovirus, Strain AD-169 • * Herpes simplex virus type 2, Strain G • * Human Coronavirus, Strain 229E • * Influenza A virus/Hong Kong Strain • * Influenza B virus, Strain B/Hong Kong/5/72 • * Respiratory syncytial virus (RSV), Strain Long • * Rotavirus Strain WA • * Kills Pandemic 2009 H1N1 Influenza A virus

BLOODBORNE PATHOGENS: * Hepatitis B virus (HBV)-Duck HBV • * Hepatitis C virus (Human) (HCV)-Bovine Diarrhea virus • * HIV-1 (AIDS virus) Strain HTLV-II

PATHOGENIC FUNGI: *Candida albicans*
Mycobacterium (TB): *Mycobacterium bovis* - BCG (TB)

BACTÉRIES • *Bordetella bronchiseptica* • *Bordetella pertussis* • *Burkholderia cepacia* • *Campylobacter jejuni* • *Enterobacter aerogenes* • *Escherichia coli* • *Escherichia coli* 0157:H7 • *Klebsiella pneumoniae* • *Listeria monocytogenes* • *Proteus vulgaris* • *Pseudomonas aeruginosa* • *Salmonella enterica* • *Serratia marcescens* • *Shigella dysenteriae* • *Staphylococcus aureus* • *Streptococcus pyogenes* • *Vibrio cholera* • *Yersinia enterocolitica*

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BACTERICIDAL, TUBERCULOCIDAL, VIRUCIDAL* IN 3 MINUTES
BACTÉRICIDE, TUBERCULOCIDE, VIRUCIDE* EN 3 MINUTES

ACTIVE INGREDIENTS: n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl ethylbenzyl ammonium chlorides... 0.14%
OTHER INGREDIENTS: n-Alkyl (60% C₁₂, 30% C₁₄, 5% C₁₆, 5% C₁₈) dimethyl benzyl ammonium chlorides... 0.14%
TOTAL... 99.72%
TOTAL... 100.00%

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(01)00310819001007

KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED SURFACES OF ULTRASOUND TRANSDUCERS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in three (3) minutes at room temperature (68°-77°) in healthcare or other settings in which there is an expected likelihood of soiling of reusable surface/objects with blood or body fluids, and in which the surface/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus type 1 (HIV-1) (associated with AIDS), Hepatitis B virus (HBV) and Hepatitis C virus (HCV).

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DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling. To Dispense Wipes: Remove lid and discard inner seal from canister. Find center of wipe roll, remove first wipe for use. Twist center of next wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Repeat lid. Dispose remaining wipes as necessary by pulling out at an angle. When not in use keep canister lid lid closed to prevent microbial loss.

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PERSONAL PROTECTION: When using this product, wear disposable protective gloves, protective gown, face mask, and eye coverings when handling HIV-1 (AIDS virus), HBV or HCV infected blood/body fluids. Cleaning Procedure: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal cloth. Open, and unfold first germicidal cloth to remove heavily soiled. Dispose of Infectious Materials: Dispose of used towel/linen in accordance with local regulations for infectious waste disposal. Contact Time: Use second germicidal cloth to thoroughly wet surface. Allow to remain wet three (3) minutes. Let air dry. Although efficacy at a 30 second contact time has been shown to be adequate against HIV-1 (AIDS VIRUS), this time is not sufficient for the other non-HIV organisms listed on this label. Therefore, a 3 minute wet contact time must be used for all listed non-HIV organisms.

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