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Bleach

LARGE WIPE
GRANDE LINGETTE

PROFESSIONAL AND HOSPITAL CLEANER AND DISINFECTANT
BACTERICIDAL, FUNGICIDAL, TUBERCULOCIDAL, AND VIRUCIDAL IN 4 MINUTES

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• *Mycobacterium bovis*-BCG (TB)

8AP24300

Disinfectant cleaner for general cleaning and disinfecting hard, nonporous environmental surfaces in healthcare facilities. **DIRECTIONS FOR USE:** To dispense: Remove lid and discard inner seal from container. Find center of wipe roll, remove first wipe for use, twist corner of next wipe into a point and thread through the hole in the container lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out at an angle. When not in use keep center cap of lid closed to prevent moisture loss. **To clean, disinfect and deodorize:** Use a wipe to remove heavy soil. Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for 4 minutes. Use additional wipes if needed to assure continuous 4 minute wet contact time. **To Clean or Remove Stains:** Wipe surface with wipe; let air dry. **Special Instructions for disinfection against Clostridium difficile spores: Personal Protection:** Wear appropriate barrier protection such as gloves, gowns, masks and eye covering. **Cleaning Procedure:** Fecal matter/waste must be thoroughly cleaned from surfaces/objects before disinfection by application with Sani-Cloth Bleach Germicidal Disposable Wipe. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right, on horizontal surfaces, and top to bottom, on vertical surfaces, to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths. **Infectious Materials Disposal:** Materials used in the cleaning process that may contain feces/waste are to be disposed of immediately in accordance to local regulations for infectious materials disposal. **Special Instructions for disinfection against HIV-1, Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV) of surfaces soiled with blood/fluids: Personal protection:** Wear appropriate barrier protection such as gloves, gowns, masks and eye covering when handling infected blood or body fluids. **Cleaning procedure:** All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal wipe. Open, unfold and use first germicidal wipe to remove heavy soil. **Disposal of infectious materials:** Dispose of used wipe immediately in accordance with local regulations for infectious waste disposal. **CAUTION:** May cause skin and eye irritation. Avoid contact with eyes and skin. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet. Sodium hypochlorite may cause damage to aluminum surfaces. **FIRST AID:** Contains sodium hypochlorite. If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present after the first 5 minutes, then continue rinsing. **STORAGE:** Store at room temperature. Store tightly closed in a cool dark place in original container away from sunlight. Do not freeze. **DISPOSAL:** Towellette Disposal: Do not reuse towellette. Dispose of used towellette in trash. Do not flush in toilet. Container Disposal: Non-refillable container. Do not reuse or refill this container. Offer for recycling. If recycling is not available, put in trash collection.

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Manufactured by / Fabriqué par : Professional Disposables International, Inc.
Imported by / Importé par : Professional Disposables International, Ltd.