PDI®**SANI-CLOTH® PLUS**
GERMICIDAL DISPOSABLE CLOTH / SERVIETTE GERMICIDE JETABLE

Bactericidal • Virucidal* • Cleans and Disinfects

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- Effective against Staphylococcus aureus, Salmonella choleraesuis, and Pseudomonas aeruginosa in 5 minutes.
- Kills Methicillin resistant Staphylococcus aureus (MRSA) in 5 minutes.
- Kills Vancomycin resistant Enterococcus faecalis (VREF) in 5 minutes.
- Kills Escherichia coli (E. coli) O157:H7 in 5 minutes.
- *Kills Human Respiratory Syncytial (HRS) virus in 5 minutes.
- *Kills Influenza A2/Hong Kong and Herpes Simplex Type 2 in 5 minutes.
- *Kills HCV (Hepatitis C Virus) in 5 minutes.
- *Kills HIV-1 on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in 30 seconds at room temperature (20 - 25 degrees C) in health care or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1).

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ACTIVE INGREDIENTS:

n-Alkyl (68% C₁₂, 32% C₁₄) dimethyl ethylbenzyl ammonium chlorides.....0.125%

n-Alkyl (5% C₁₂, 60% C₁₄, 30% C₁₆, 5% C₁₈) dimethyl benzyl ammonium chlorides.....0.125%

Isopropyl Alcohol.....14.850%

**KEEP OUT OF REACH OF CHILDREN
CAUTION** See back panel for additional precautionary statements**INGRÉDIENTS ACTIFS :**

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A non woven disposable cloth for use in hospitals and other critical care areas where control of the hazards of cross-contamination is required. Use on surfaces and equipment such as stainless steel, Formica, glass tables, carts, baskets, counters, cabinets, telephones and other hard non-porous surfaces.

DIRECTIONS FOR USE:

DISPENSER: To start feed: Remove large cover and discard seal from container. From center of towelette roll, pull up towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and snap off at a 90 deg. angle. Remaining towelettes feed automatically ready for next use. When through using, keep small center cap closed to prevent moisture loss.

Open, unfold wipe to remove gross filth. Use a second wipe to thoroughly wet surface. Allow to remain wet 5 minutes where required. Let air dry.

***SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS**

PERSONAL PROTECTION: When using the germicidal cloth, wear disposable protective gloves, protective gowns, face masks, or eye coverings as appropriate when handling HIV-1 infected blood or body fluids.

CLEANING PROCEDURE: All blood and other body fluids must be thoroughly cleaned from surface and objects before disinfection by the germicidal cloth. Open, unfold first germicidal cloth to remove gross filth.

CONTACT TIME: Use second germicidal cloth to thoroughly wet surface. Allow to remain wet 30 seconds, let air dry. Although efficacy at a 30 second contact time has been shown to be adequate against HIV-1 this time would not be sufficient for other organisms. Use listed disinfection times against all of the organisms claimed.

DISPOSAL OF INFECTIOUS MATERIALS: Dispose of used towelette in accordance with local regulations for infectious waste disposal.

PRECAUTIONS:

CAUTION: Avoid contact with eyes.

FIRST AID: If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

STORAGE AND DISPOSAL: Towelette: Do not reuse towel. Dispose of used towel in trash. Dispenser: Do not reuse empty container. Wrap container and put in trash collection.

DO NOT USE OR STORE NEAR HEAT OR OPEN FLAME

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Professional Disposables International, Ltd.