Sani-24[®]

GERMICIDAL SPRAY

Use on hard, nonporous environmental surfaces.
Use for soft surface sanitization.

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GENERAL GUIDELINES FOR USE INSTRUCCIONES GENERALES DE USO

1. Hold grip section of bottle under spray nozzle.

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3. If present, remove heavy soil loads prior to disinfecting.

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4. Position spray bottle toward untreated surface. Using index finger, squeeze trigger to dispense liquid.

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5. To Disinfect: Spray Sani-24[®] Germicidal Spray 6-8 inches from surface. Ensure even coverage and thorough wetness. Let stand for one (1) minute.

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6. For Continuously Active Disinfection (CAD) for 24 Hours: Spray Sani-24 Germicidal Spray 6-8 inches from surface. Ensure even coverage and thorough wetness. Allow to air dry. Use of this product should not alter standard cleaning and disinfection practices. If surface treated with Sani-24 Spray is subsequently cleaned with anything other than Sani-24 Spray, Sani-24 Spray must be reapplied for Continuously Active Disinfection.

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7. Soft Surface Sanitization: Hold container 6-8 inches from surface until thoroughly wet. Let stand for 10 seconds. Let air dry.

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Refer to device manufacturer's instructions for use prior to using.

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©2020 PDI PDI01201059
400 Chestnut Ridge Road, Woodcliff Lake, New Jersey 07677
800.999.6423 pdihc.com

PATIENT CARE

INTERVENTIONAL CARE

ENVIRONMENT OF CARE

+ PDI[®]
BE THE DIFFERENCE[™]