

General Guidelines For Use

Instrucciones Generales De Uso

Sani-HyPerCide™
GERMICIDAL
DISPOSABLE WIPE
Toallitas Germicidas Desechables



Use on hard, nonporous environmental surfaces.

Uselas en superficies ambientales duras y no porosas.



1. Put on PPE. Always dispense wipe through lid. Find center of wipe roll, twist corner of wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Replace lid.

Siempre extraiga la toalla a través de la tapa. Busque el centro del rollo de toallas, doble la esquina de la toalla en punta y extráigala a través de la abertura de la tapa del envase. Extraiga aproximadamente una pulgada. Vuelva a colocar la tapa.



2. Cover the opening half way with one hand. Remove wipe with a uniform pull away from face and eyes. When not in use, keep lid closed to prevent moisture loss.

Cubra la mitad de la abertura con una mano. Extraiga la toalla con un tirón uniforme, lejos de la cara y los ojos. Cuando no esté usando el producto, mantenga la tapa cerrada para evitar la pérdida de humedad.




- 3a. If present, remove visible soil loads prior to disinfecting.

En caso de presencia de mucha suciedad, límpiela antes de desinfectar.



- 3b. Unfold a clean wipe and thoroughly wet surface.

Abra una toalla limpia y humedezca bien con ella la superficie.

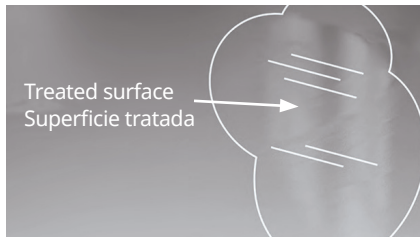

4. Allow surface to remain visibly wet for one (1) minute. For *C. diff* spores, allow surface to remain visibly wet for five (5) minutes. Let air dry.

Deje que la superficie tratada permanezca húmeda por un (1) minuto. Para eliminar las esporas de *C. diff*, deje que la superficie permanezca visiblemente mojada por cinco (5) minutos.



5. Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet.

No reutilice la toallita. Deséchela en el basurero. No la deseche en el inodoro.

