

General Guidelines For Use

Instrucciones Generales De Uso

Sani-HyPerCide[®]
GERMICIDAL
DISPOSABLE WIPE
Toallitas Germicidas Desechables



Use on hard, nonporous environmental surfaces.

Uselas en superficies ambientales duras y no porosas.



1. Always dispense wipe through lid. Find center of wipe roll, remove first wipe for use, thread next wipe through the hole in the canister lid. Pull through at least one inch. Replace lid. To secure wipe, pull wipe down into small opening.

1. Siempre extraiga la toalla a través de la tapa. Busque el centro del rollo de toallas, saque la primera toalla que va a usar y después inserte la siguiente toalla a través de la abertura de la tapa del envase. Extraiga, al menos, una pulgada. Vuelva a colocar la tapa. Para asegurar la toalla, empújela al interior de la pequeña abertura.



2a. Cover the opening half way with one hand. Remove wipe(s) with a uniform pull away from face and eyes.

2b. Dispense single wipes as necessary by pulling out at an angle through the small opening.

2c. Dispense multiple wipes as necessary by pulling vertically through the large opening. Pull down into small opening to tear.

2a. Cubra la mitad de la abertura con una mano. Extraiga las toallas con un tirón uniforme alejado del rostro y los ojos.

2b. Dispense toallas individualmente según se necesiten tirando de ellas a un ángulo a través de la pequeña abertura.

2c. Dispense varias toallas según se necesiten tirando de ellas verticalmente a través de la abertura grande. Empújelas hacia la pequeña abertura para rasgarlas.




3. If present, use a wipe to remove visible soil prior to disinfecting.

3. Antes de desinfectar, use una toalla para eliminar la suciedad visible si existiera.



4. Unfold a clean wipe and thoroughly wet surface.

4. Abra una toalla limpia y humedezca bien con ella la superficie.

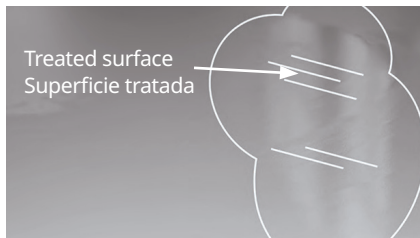

5. Allow treated surface to remain wet for one (1) minute. If treating for *C. diff* allow surface to remain wet for five (5) minutes. Let air dry.

5. Deje que la superficie tratada permanezca húmeda por un (1) minutos. Si trata *C. diff*, permita que la superficie permanezca húmeda durante cinco (5) minutos. Deje secar al aire.



6. Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet.

6. No reutilice la toallita. Deséchela en el basurero. No la deseche en el inodoro.



©2023 PDI 1023 UPDATE PDI05203062
400 Chestnut Ridge Road, Woodcliff Lake, New Jersey 07677
800.999.6423 pdihc.com

PDI[®]
BE THE DIFFERENCE