

+Prevantics®
3.15% Chlorhexidine Gluconate (CHG) and
70% Isopropyl Alcohol (IPA)

Device
SWAB



GENERAL GUIDELINES FOR USE

For the disinfection of blood culture bottle tops follow the steps below



1. Look for the words “tear below” (found on back side of packet) and tear downward diagonally to retrieve swab.




2. Prior to access, apply swab to the top of the blood culture collection bottle by holding the swab between the thumb and the index finger and scrub using repeated back and forth strokes for 5 seconds.



3. Allow to air dry for 5 seconds. Do not blot or wipe dry.



4. Discard after single use.

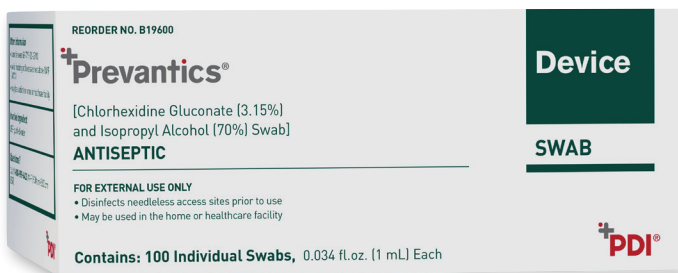

SCAN HERE
TO WATCH VIDEO

+PDI®
BE THE DIFFERENCE