


Sani-Cloth® Wipes Materials Compatibility Reference Guide

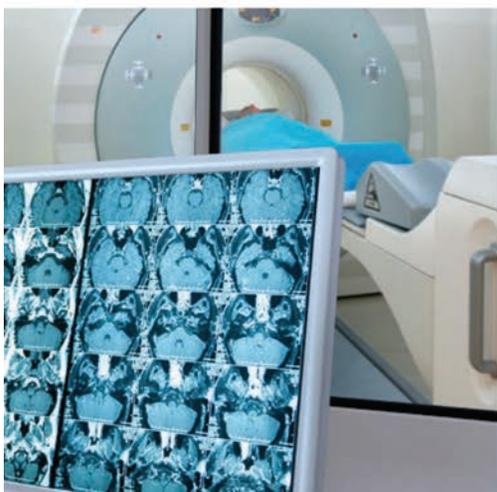


COMPATIBLE
by design™ 


PDI
BE THE DIFFERENCE

PDI's Compatibility Approach.

Our scientists use a variety of ASTM and ISO material testing standards to understand the effects of our disinfectants on common material surfaces.

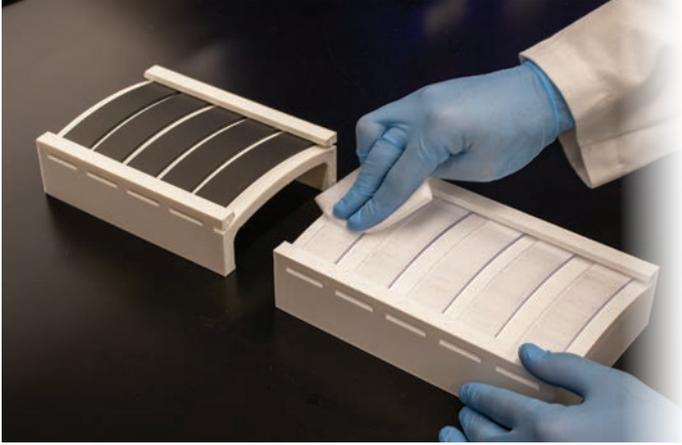

Wipe test:

Material surfaces were wiped 120 times with a 30-minute dry period between each wipe. Simulates time-dependent material exposure in a healthcare setting.

Wipe test under constant strain:

Materials were wiped 120 times with a 30-minute dry period between each wipe under 1% constant strain. Simulates time-dependent material exposure under strain in the healthcare setting.



Immersion test:

Coupons of the materials were wrapped inside the disinfectant wipe and placed inside Ziploc-type bags. Three times each week, the test samples were unwrapped and inspected for deleterious effects. Observations were noted and fresh wipes were used to wrap the test surfaces and continue the exposure. This method examines the durability of the material under extreme chemical exposure.

We have partnered with industry leaders in raw material manufacturing for medical devices to evaluate our broad family of products on thermoplastic chemical resistance.^{1,2,3,4,5}

PDI's Compatible by Design™ System.

Legend



No visible surface change (damage/degradation) to the material is likely to occur when used according to the directions for use.



Some visible effects or cosmetic changes such as streaking, discoloration, or clouding, may be observed with long-term exposure.



Visible surface change (damage/degradation) may occur with long-term exposure.

The challenge: equipment damage.

Incompatible cleaning and disinfection products in healthcare creates dangerous safety issues for patients, costs healthcare facilities millions of dollars annually, and impedes effective cleaning and disinfection.


Biomedical engineers estimate a 20-27% equipment life span reduction due to incompatible disinfectants.⁹



Types of surface change.

Exposure to disinfectants can lead to degradation of surfaces and, for plastics, environmental stress cracking (ESC).

Discoloration

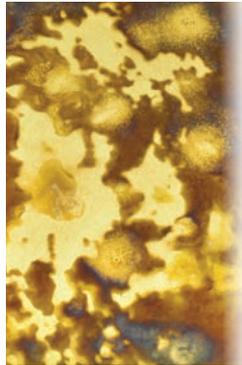


Material's surface changes color due to environmental conditions. Often described as fading, yellowing, staining, darkening, bleaching, spotting, etc.

Residue



Material remaining on the surface after the solvent or additives have evaporated. Residue is unsightly but often removable.

Cracking

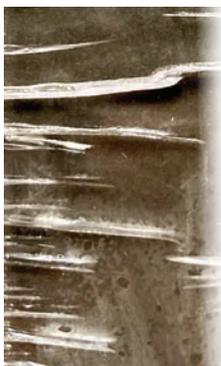

Material failure, fracture, crack, and crazing described as localized deformation occurring in a material.

Corrosion



Gradual deterioration of materials caused by a chemical reaction on the surface. Can lead to pitting, rusting, deformation, etc.

Delamination



Mode of failure where a material fractures into layers along a plane parallel to a surface.

Guidance from the CDC, FDA and Joint Commission.

“Disinfect noncritical surfaces and medical devices with an EPA-registered hospital disinfectant using the label’s safety precautions and use directions.”¹⁰

- CDC from “Guideline for Disinfection in Healthcare facilities”

“FDA recommends that you validate your disinfection processes and instructions. FDA also recommends that you follow the recommendations in device-specific FDA guidance documents or any relevant FDA-recognized standards.”¹¹

- FDA from “Reprocessing Medical Devices in Health Care Settings: Validation Methods and Labeling”



The goal of our Compatible by Design Program is to help your facility meet the requirements of your accreditation organization.



Common surface materials found in a patient room.



PDI's Compatibility Search Tool.

Healthcare Professionals.

Do you know your medical devices' disinfection guidelines? We can help identify which products are approved and compatible!

[CLICK HERE TO RECEIVE MORE INFORMATION](#)



Check your Equipment Compatibility.

Select a PDI Product.

Manufacturer:

Equipment Type:

Equipment Item:

PDI Tested Product:

Select



Select

Select

All Tested Products

CHECK COMPATIBILITY

We're here to help. Check our [Resource Library](#) for additional compatibility information.



Scan here to learn more about PDI's Compatibility Search Tool or go to <https://pdihc.com/compatibility-hcp>

PDI's compatible disinfectants.

Sani-HP1™ Germicidal Disposable Wipe



One minute, one step, one smart choice for daily disinfection—innovative hydrogen peroxide formulation with excellent surface compatibility and no PPE required.

Sani-HyPerCide® Germicidal Disposable Wipe



A powerful, ready to use hydrogen peroxide formula designed to address HAI-causing microorganisms, including *Clostridioides difficile*, without compromising compatibility.

Super Sani-Cloth® Germicidal Disposable Wipe



Ideal for daily use in fast-paced environments that require a short contact time and broad coverage of microorganisms.

Sani-Cloth® Prime Germicidal Disposable Wipe



Powered by a next generation formulation effective against HAI causing pathogens in just one minute.

Sani-Cloth® Bleach Germicidal Disposable Wipe



Ideal for disinfecting areas contaminated with *Clostridioides difficile* spores, *Candida auris*, and Norovirus.

Sani-Cloth® AF3 Germicidal Disposable Wipe



Alcohol-free. Fragrance-free. Worry-free. Ideal for use around patients and staff, especially those with respiratory sensitivities.

Sani-Cloth® Plus Germicidal Disposable Cloth



Simple, reliable protection— one-step disinfection ideal for physician offices, dental clinics, and other alternate care settings.

COMPATIBLE by design™

Educational and point-of-care accessories.

Developed to assist you in facility protocol compliance.



PDI Sani-Bracket® for wall/pole mounting of a canister



Sani-Canister Caddy® for placement on flat surfaces (bilingual)



Sani-TAG® Equipment ID System for identification of equipment to ensure the proper wipe is used on a particular piece of equipment



Educational Wall Charts and signage (bilingual)

Order information.

REORDER NO. WIPE SIZE CASE PACK CASE WGT CASE CUBE PALLET TI/HI



Sani-HP1™ Germicidal Disposable Wipe

1 minute, broad spectrum protection for your patients, staff and surfaces. Ideal for daily disinfection.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Medium Canister	P62572	6" x 5"	12/275s	27.86 lbs	1.430 ft	10/3
Large Canister	P42672	6" x 9"	12/165s	29.78 lbs	1.430 ft	10/3
X-Large Canister	P83184	7.5" x 15"	6/95s	17.82 lbs	0.936 ft	10/4
Bulk	P0125P	11" x 12"	2/300s	20.41 lbs	0.853 ft	10/4



Sani-HyPerCide® Germicidal Disposable Wipe

A ready to use hydrogen peroxide formula designed to address HAI causing microorganisms, including *Clostridioides difficile* without compromising compatibility.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Large Canister	P27372	6" x 6.75"	12/160s	23.58 lbs	1.430 ft	10/3
X-Large Canister	P26584	7.5" x 15"	6/65s	15.10 lbs	0.936 ft	10/4



Super Sani-Cloth® Germicidal Disposable Wipe

2 minute contact time allows for a quick room turnover. Ideal for daily use in fast-paced environments that require short contact times and broad coverage of microorganisms.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Large Canister	Q55172	6" x 6.75"	12/160s	25.63 lbs	1.430 ft	10/3
X-Large Canister	P86984	7.5" x 15"	6/75s	16.71 lbs	0.936 ft	10/4
Large Individual Packet	H04082	5" x 8"	10/50s	7.72 lbs	0.539 ft	20/4
X-Large Individual Packet	U87295	11.5" x 11.75"	3/50s	7.51 lbs	0.500 ft	24/3
Softpack	A22480	8.2" x 9.8"	9/80s	15.40 lbs	0.948 ft	10/4



Sani-Cloth® Prime Germicidal Disposable Wipe

Powered by a next generation formulation, help protect your patients, staff, and facility from HAIs in just 1 minute.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Large Canister	P25372	6" x 6.75"	12/160s	30.56 lbs	1.430 ft	10/3
X-Large Canister	P24284	7.5" x 15"	6/70s	18.50 lbs	0.936 ft	10/4
Large Individual Packet	H06182	5" x 8"	10/50s	9.40 lbs	0.636 ft	17/4
X-Large Individual Packet	U13195	11.5" x 11.75"	3/50s	9.00 lbs	0.500 ft	24/3



Sani-Cloth® Bleach Germicidal Disposable Wipe

Ideal for disinfecting areas contaminated with *Clostridioides difficile* spores and Norovirus.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Clinical Size Wipes Canister	P84172	6" x 5"	12/160s	26.96 lbs	1.430 ft	10/3
Large Canister	P54072	6" x 10.5"	12/75s	26.59 lbs	1.430 ft	10/3
X-Large Canister	P25784	7.5" x 15"	6/65s	19.85 lbs	0.936 ft	10/4
Large Individual Packet	H58195	5" x 7"	10/40s	8.10 lbs	0.505 ft	19/5
X-Large Individual Packet	U26595	11.5" x 11.75"	3/40s	8.20 lbs	0.500 ft	24/3
Pail	P7007P	7.5" x 15"	2/160s	16.50 lbs	0.953 ft	10/5
Refill (for Pail)	P700RF	7.5" x 15"	2/160s	15.20 lbs	0.529 ft	18/3



Sani-Cloth® AF3 Germicidal Disposable Wipe

Helping to protect your staff, patients, and residents has never been easier! Alcohol and fragrance-free formula is ideal for use around those with respiratory sensitivities.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Large Canister	P13872	6" x 6.75"	12/160s	26.14 lbs	1.430 ft	10/3
X-Large Canister	P72584	7.5" x 15"	6/75s	16.80 lbs	0.936 ft	10/4
Large Individual Packet	H59200	5" x 8"	10/50s	7.99 lbs	0.539 ft	20/4
X-Large Individual Packet	U27500	11.5" x 11.75"	3/50s	7.60 lbs	0.500 ft	24/3
Bulk	P938250	11" x 12"	2/250s	20.03 lbs	0.853 ft	10/4



Sani-Cloth® Plus Germicidal Disposable Wipe

Ideal for use in alternate care settings, including physician and dental offices.

Canister	Reorder No.	Wipe Size	Case Pack	Case Wgt	Case Cube	Pallet TI/HI
Large Canister	Q89072	6" x 6.75"	12/160s	26.14 lbs	1.430 ft	10/3
X-Large Canister	Q85084	7.5" x 15"	6/65s	14.85 lbs	0.936 ft	10/4



Scan here to learn more about PDI's compatibility partnerships or go to <https://pdihc.com/equipment-compatibility>

¹Data on file. Compatibility study 8-23.2017.

²Zettel, M., Liu, Y., Lamont, M., Bernhard, K. and Hicks J., October 2022. "Standard test method for evaluation of chemical resistance of thermoplastics for non-disposable medical devices- a voice from industry collaboration through the COVID-19 pandemic". Medical Design Briefs. <https://www.medicaldesignbriefs.com/component/content/article/mdb/pub/features/articles/46685>.

³Esposito, A. October 2016. "PDI and SABIC join forces to test medical devices to improve patient protection from infection". Medical Design & Outsourcing. <https://www.medicaldesignandoutsourcing.com/pdi-sabic-join-forces-test-medical-device-plastics-help-improve-patient-protection-infection/>

⁴Ratnagiri, R., and Martin, R. (2022). Resistance of DuPont polymers to disinfecting chemicals, including those effective against the SARS-CoV-2 virus. [White paper], Dupont. <https://www.dupont.com/content/dam/dupont/amer/us/en/mobility/public/documents/en/Home-Appliance-Whitepaper-FNL.pdf>

⁵(2020). Covestro Polycarbonates: Compatibility with Disinfectants used against SARS-CoV-2. [White paper], Covestro. <https://solutions.covestro.com/-/media/covestro/solution-center/whitepapers/cov-chemical-compatibility-to-disinfectants-used-against-sars-cov-2-2020-06.pdf>

⁶Marlite® is a registered trademark of Marlite, Inc.

⁷Tritan™ is a trademark of Eastman Chemical Company.

⁸3Form™ and Varia™ are trademarks of 3Form, LLC

⁹Data on file. Compatibility Study 8-28-17.

¹⁰<https://www.cdc.gov/infectioncontrol/guidelines/disinfection/index.html#rec5g>

¹¹<https://www.fda.gov/regulatory-information/search-fda-guidance-documents/reprocessing-medical-devices-health-care-settings-validation-methods-and-labeling>

¹²Sani-Cloth® Plus Germicidal Disposable Cloth compares to competitor quat/low-alcohol disinfectant products.

*Sani-HP1™ is not yet available for sale or distribution into Hawaii.

**On hard nonporous surfaces