

Prevent with Prevantics® Device Swab.

Prevantics Device Swab helps improve 'scrub the hub' compliance to better protect against CLABSIs.¹

Our device swab's 3.15% Chlorhexidine Gluconate and 70% Isopropyl Alcohol formula, with **mechanical friction** disinfection, and **fast 5-second scrub and 5-second dry time**², work together to **help lower CLABSI rates**.

+ Prevantics®



+ PDI®
BE THE DIFFERENCE™

Prevantics Device Swab is available in three convenient formats, including a strip format for IV poles to ensure point-of-care accessibility.

Features & Benefits

- + Conveniently designed to help disinfect needleless access sites.
- + Contains the first and only 3.15% Chlorhexidine Gluconate and 70% Isopropyl Alcohol formulation for disinfecting needleless access sites.
- + Quick 5 second scrub time and 5 second dry time helps with staff compliance.
- + Achieves a $>4.0 \log_{10}$ reduction against 7 tested clinically relevant microorganisms known to cause HAIs:^{2,3}

- *Candida albicans*
- *Escherichia coli*
- *Pseudomonas aeruginosa*
- Methicillin Resistant *Staphylococcus aureus*
- *Candida parapsilosis*
- *Staphylococcus aureus*
- *Staphylococcus epidermidis*

How to Use


Familiar, easy to use prep pad format.

Step 1




Tear down diagonally.

Step 2

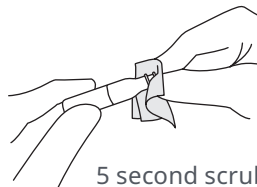

5 second scrub time.

Step 3

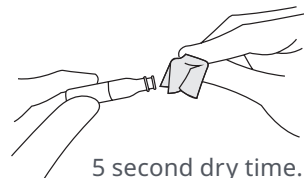

5 second dry time.



REORDER NO. CASE PACK CASE WGT CASE CUBE PALLET TI/HI

Prevantics® Device Swab and Strip

	REORDER NO.	CASE PACK	CASE WGT	CASE CUBE	PALLET TI/HI
Swab 1.0 mL	B19600	100/box, 10 boxes/case	4.30 lbs	0.28 ft	21/7
Swab Strip 1.0 mL	B123ST	160/box, 10 boxes/case	4.68 lbs	0.52 ft	12/7
Large Swab 1.2 mL	C71200	150/box, 10 boxes/case	7.12 lbs	0.48 ft	12/8

Learn more



¹Central line-associated bloodstream infections.

²PDI Study BSLI 140304-250 July 23, 2014.

³CDC HAI Prevalence Survey. Magill, S.S., Edwards, J.R., Bamberg, W., et al. Multistate Point Prevalence Survey of Healthcare-Associated Infections. N Engl J Med 2014;370:1198-208.