

Super Sani-Cloth®

GERMICIDAL DISPOSABLE WIPE

Use on hard, nonporous environmental surfaces.

Uselas en superficies ambientales duras y no porosas.



GENERAL GUIDELINES FOR USE

INSTRUCCIONES GENERALES DE USO



- 1. Open package lid.**
Abra la tapa del paquete.



- 2. Completely remove inner seal.**
Retire por completo el precinto interno.



- 3. Pull out wipe at angle through opening.**
Saque una toalla por el agujero, en diagonal.



- 4. Snap lid closed after use to retain moisture.**
Cierre la tapa a presión después del uso para conservar la humedad.



- 5a. If present, remove visible soil prior to disinfecting.**
En caso de presencia de suciedad, límpiela antes de desinfectar.



- 5b. Unfold a clean wipe and thoroughly wet surface.**
Abra una toalla limpia y humedezca bien con ella la superficie.



- 6. Allow surface to remain visibly wet for two (2) minutes. Let air dry.**
Deje que la superficie permanezca húmeda por dos (2) minutos. Deje que la superficie se seque al aire.



- 7. Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet.**
No reutilice la toallita. Deséchela en el basurero. No la deseche en el inodoro.

Visit pdihc.com for more info.

©2021 PDI02217689
400 Chestnut Ridge Road, Woodcliff Lake, New Jersey 07677
800.999.6423 pdihc.com



INTERVENTIONAL CARE PATIENT CARE ENVIRONMENT OF CARE